


# Organiser for Carbon Ultralight

SMALL BAG FOR YOUR NECESSITIES

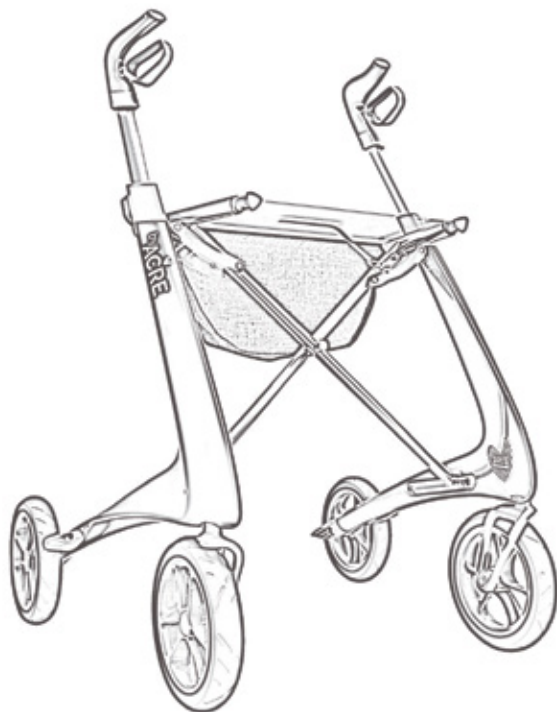

Congratulations with your new organiser for the byACRE Carbon Ultralight rollator. The organiser is a smaller bag placed on the "inside" of the rollator so you easily can carry smaller items you use frequently. Please read this instruction in order to get the most out of your organiser.





by **ACRE**

# Use of the organiser

The organiser is easy accessed by opening the zipper. Remember to lock the zipper when you walk to ensure that you are not losing any items on your way.


- ⚠ Do not fill the organiser more than it can still be closed.
- ⚠ Do not overload the organiser.

## Attachment

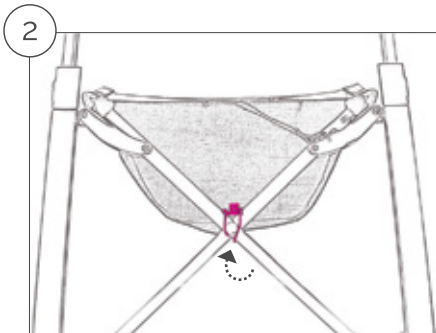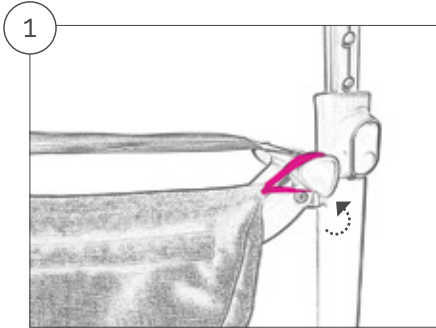


1. Fold the Carbon Ultralight rollator. It is easiest to attach the organiser when the Carbon Ultralight rollator is partly folded. Attach the organiser by placing the two flat loops on the side of the organiser around the thinner cylindrical part of the rear attachment points on the Carbon Ultralight rollator. They are placed in extension of the bars supporting the seat. Ensure that the zipper is facing away from the seat.
2. Put the centre string loop around the centre of the folding mechanism on the Carbon Ultralight rollator and engage the push button. Unfold the Carbon Ultralight rollator. The organiser is now securely attached to the Carbon Ultralight rollator and ready to use.

## Detachment

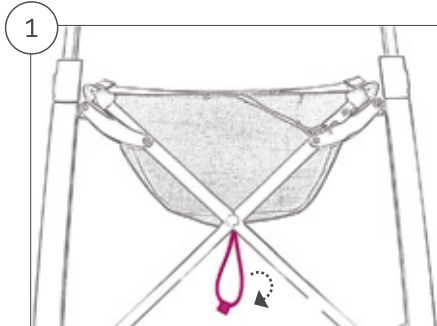

1. Fold the Carbon Ultralight rollator. Detach the push button on the centre loop string.

2. Detach the two flat side loops from the attachment points on the Carbon Ultralight rollator. Unfold the rollator if needed.

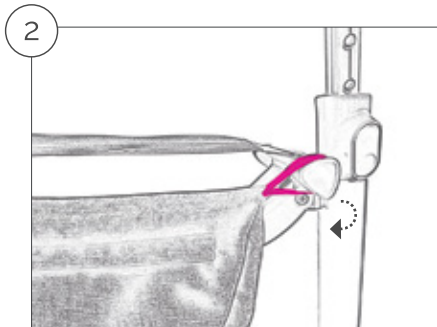

## Technical data

Material: 100% polyester

Width: 450mm

Depth: 105mm

Height: 185mm

Weight: 90g

Load capacity: 2kg

## Maintenance

The clutch can be hand washed.

## Warranty

The clutch comes with 2 years warranty against manufacturing errors. This warranty does not cover wear due to normal use.

Designed in Denmark by:

byACRE ApS

Bygmarken 4

3520 Farum

Denmark